

RRS®

RRS® HA EYES

Enhance appearance of eyes contour - Superficial wrinkles around the eyes - Dark circles & eyes tired look

RRS® HA EYES is specially formulated for ageing skin around the eyes, since the biorevitalizing filler effect of Hyaluronic Acid is assisted by a specific biorevitalisation solution. The effects of non cross linked Hyaluronic Acid on wrinkles are enhanced by a specific biorevitalisation solution containing vitamins, antioxidants, microcirculatory and blending/bleaching actives.

No limit of age**

CE
1014 **Class III medical device**

*Hyaluronic Acid is from biotechnological, non animal origin

**No studies are available for use during pregnancy or minors under 18.



box of 2 ampoules / 12 ampoules / 24 ampoules
containing 1,5 ml (0,05 Fl.oz.)

NON CROSS LINKED HA + BIOREVITALIZING SOLUTION

Intradermal implant that contains 25 mg of a non cross linked, resorbable Hyaluronic Acid* associated to a biorevitalization solution and specially suitable for eyes contour dermal filling and biorevitalization.

For Eyes

RRS[®]



RRS[®] HA Eyes must be used under appropriate aseptic conditions in an authorized clinic on healthy, disinfected skin.

Area: eyes

Average volume/session: 0.65 ml area / 1.30 ml both eyes

Type of injection: micro dermal papule

Frequency: 1 session / 2 weeks (1 protocol = 4 sessions average)

Recommended number of sessions: repeat protocol 1-3 times a year

Combination with other aesthetic treatments:

- RRS[®] can enhance results of other medical treatments or devices
- Botulinum toxin: two weeks before/after RRS[®] injection
- Microneedling device: RRS[®] can be injected immediately before
- Lasers: RRS[®] injection can be done immediately before

RRS[®] HA EYES

Before the treatment the physician should inform the patient about:

- indications and effects
- possibilities of the side effects (pain, redness, ecchymosis, stinging sensations and swelling, local inflammation, usually disappearing in 24 hours)
- check allergy test

Sensitive skins may benefit from application of an anaesthetic cream prior to the treatment.

We recommend to have a consent signed by the patient.

After the treatment

Avoid: extreme temperatures, Saunas - Hammam; direct exposure to sun or UV.

From next day make-up can be used.

Contraindications

Allergy to any of the ingredients. Patients presenting any skin alteration, skin disease, infections or sequelae of streptococcal infections. Patients taking immunosuppressants, undergoing cortical therapy, with autoimmune disease history, patients with uncompensated diabetes, acute joint rheumatics, repetitive angina, and endocarditis.

*No studies are available for use during pregnancy and breastfeeding or in case of treatment on children or minors under 18.

HA, non-cross-linked associated to:

Antioxidants: Vit C, Naringin, Resveratrol, R-2 Viniferin, ϵ -Viniferin, R-Viniferin, Iso- ϵ -Viniferin, etc.

Flavonoids: Troxerutin, Rutin, Naringin, Hesperidin Methyl Chalcone, Hesperidin.


Saponins: Ruscogenin, Neoruscogenin, Escin.

Polyphenols: Resveratrol, Coumarin, etc.


Peptides: Dipeptide VW, Peptide Pal-GQPR.

Trace element: Organic Silicium.

AESTHETIC TREATMENT OF EYES

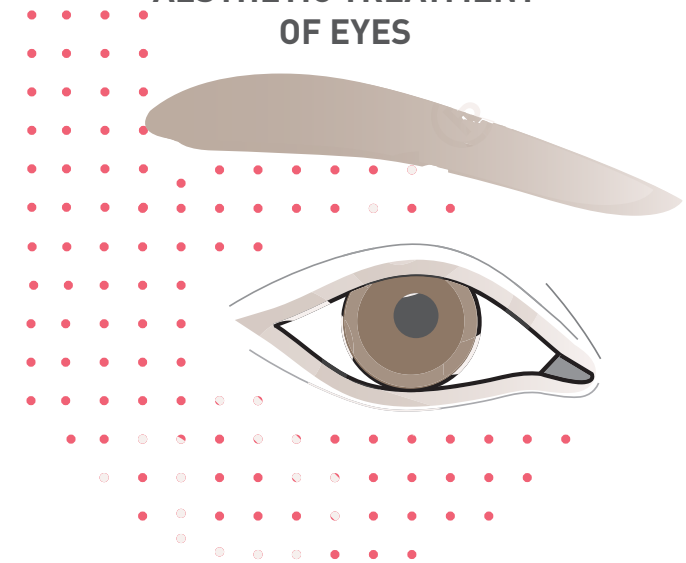

Distance between injection points (0.5 cm)

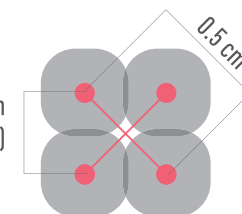

0.05ml / Injection point



Original formulations

RRS® line: the quintessence of scientific knowledge and technologic progress.

The original RRS® formulas, designed in our own R&D laboratory, are based on the purest and most effective ingredients, with scientific evidences for superficial and deep skin injections in case of skin rejuvenation, with synergetic actions on several problems.

Non cross-linked HA* from biotechnological non animal origin


Hyaluronic Acid provides:

- Antioxidant effect
- Cells migrations stimulation
- Turnover stimulation
- Matrix reorganization

The specific biorevitalization complex helps to get the following actions:

- Anti ageing
 - Dermal filling & biorevitalizing
 - Hair growth stimulation
 - Anti cellulite
 - Anti striae
 - Whitening effect
-

EFFECTS & INDICATIONS MAP: FACE

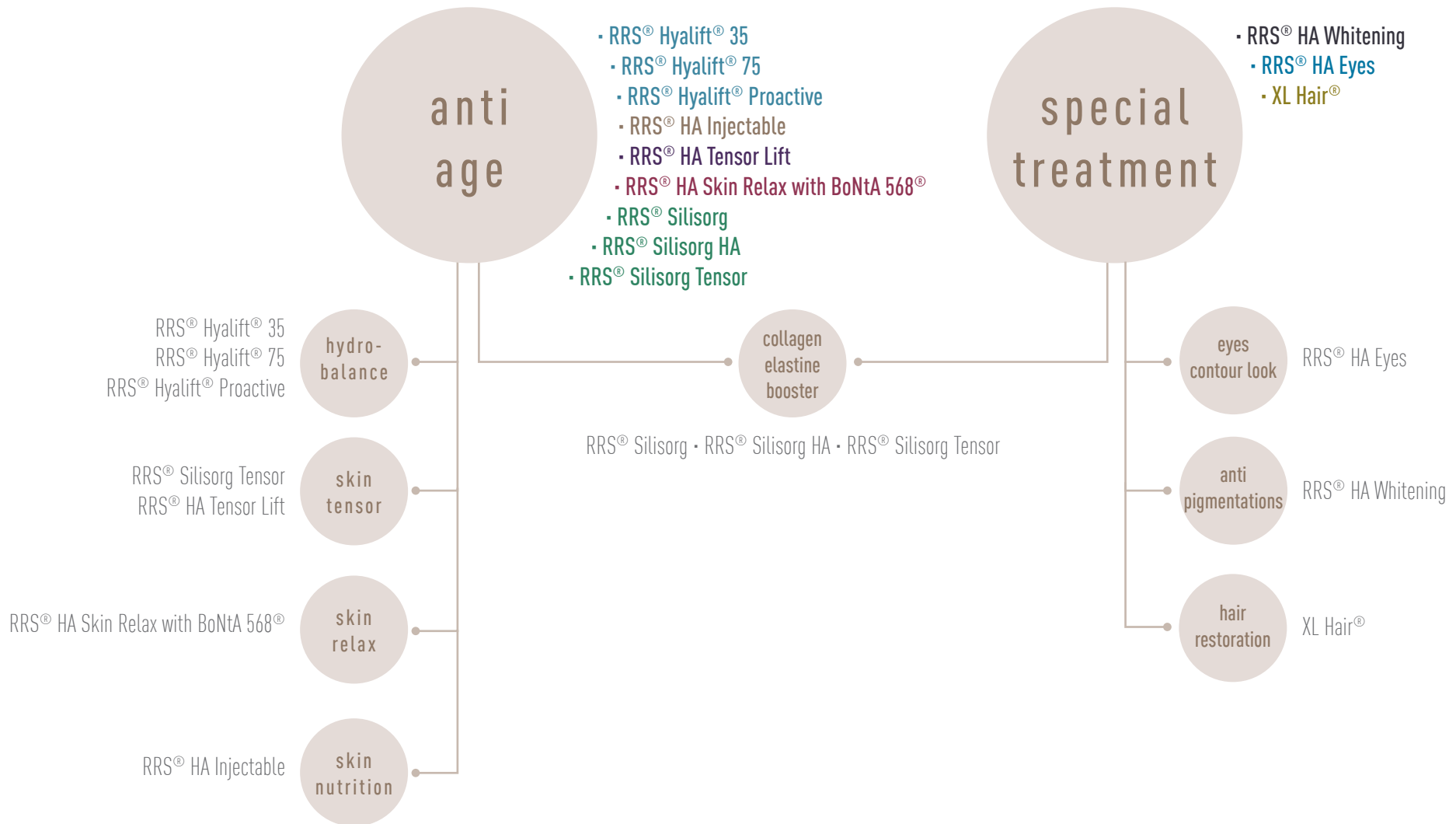

EFFECTS & INDICATIONS MAP: BODY

